Mobilex round shower stool with swivel seat Item no.302066

Montage- und Gebrauchsanleitung
Montage- og brugervejledning
User Manual
Manuale d'uso e manutenzione

MOBILEX A/S
Grønlandsvej 5
DK-8660 Skanderborg
www.mobilex.dk

**Description & intended use**

The round shower stool is a medical device which intended use is to give stable & easy seating support when disabled persons use the shower. It has an aluminium frame with height-adjustable legs and a round swivel seat.

Important safety information

Only use the device for the purpose it is intended for. Don't stand on the device. Make sure that the legs are set to the same height. Use device only in good technical condition. Do not overload the device. Use device only on flat smooth surface with all 4 legs stably placed on the surface. Always check stability of device before use.

Assembly and usage description

The shower stool is delivered separately. The package comes with 2 legs, 1 white round seat with swivel function and 5 screws, 1 light grey and 4 grey with 5 metal discs. Take the white seat and place it upside down on a table. Place the legs and be sure to place one leg under the other. Insert the light grey screw with the metal washer on the screw in the middle hole, then the other 4 screws in the same way in the 5 holes on the underside of the seat and secure the screws. Check that all the legs are firm and stable by placing the bath chair on an even surface. Check and adjust the legs to the desired height, making sure that the legs are set to the same height and all are securely locked.

Cleaning & maintenance

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives). The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary
- ✓ General retightening of the product

Recommended service life: Normally 10 years depending on use.

Storage

We recommend storage in a dry place.

Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to the list above

Disposal of the product

The product may not be disposed of with household waste but must be brought to the local recycling site.

Technical data	
Dimension of seat	Ø 29.6 cm
Seat height-levels	34.5 - 52.5 cm
Max. user weight	130 kg
Product weight	1.85 kg

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Guarantee

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

Declaration of Conformity (CE)

(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità)

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,

Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

PRODUCT LIST

Round Shower stool - item no. 302066

ACCESSORIES LIST

Item no.	Accessories item no.
302066	No

Harmonized norms used during conformity estimation:

PN-EN 12182:2012, EN 581-2:2015, PN-EN ISO 14971:2012, PN-EN 1041:2009

EU2017/745 for Medical Equipment
(über Medizinprodukte / for medicinalprodukter / del Consiglio Europeo)

In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /

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Signature/Unterschrift/Underskrift/firma

Round shower stool with swivel seat
REF 302066

MOBILEX A/S
Grønlandsvej 5
DK - 8660 Skanderborg
Tel: +45 87 93 22 20
www.mobilex.dk

5 74 0001 403350

≤130 kg

(01) 05740001403350 (11) 210921 (21) 0001 (240) 302066

2021.09.21

SN 0001

Product label for 302066

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	To be used indoor and outdoor	SN	serial no
	CE marked product	REF	Item Number
	Max load basket	MD	Medical Device
	Only to be used indoor		

Overview icon meaning